Recommendations for use

Guide to the use of your scleral lenses

This leaflet contains important information; read it and keep it for future reference. Lenses are medical devices and must always be adjusted by a contact lens specialist. For correct use, it is important that you follow all the instructions given by your specialist and all the recommendations given on the packaging of your lenses and maintenance products.
INSTRUCTIONS:

These lenses are scleral lenses for daytime wear to correct short-sightedness, long-sightedness and astigmatism on irregular corneas (keratoconus, pellucid marginal degeneration, post-refractive surgery with or without ectasia, lamellar or transiting corneal staples) or regular corneas with eye dryness and/or discomfort with hard corneal lenses.

A scleral lens is larger than a traditional hard lens and rests on the sclera (the white of the eye). After using a solution without a preservative to fill the gap between the scleral lens and the cornea, the lens-cornea combination will act in refractive terms, just like a normal cornea. Vision is thus considerably improved.

RECOMMENDED RENEWAL:

Every year

PACKAGING:

The lenses are packaged in a bottle containing a decontaminant solution.

If packaging is damaged, do not use the lens.

The label provides the following information:

	The lens brand name
	The name of its material
	The name and address of the manufacturer
	
	The batch number
	
	European conformity with health and safety requirements and certifying organisation number
	
	Expiry date
	Name of the storage solution
	Leaflet consultation

PRECAUTIONS FOR USE

Your specialist will give you precise instructions on how to look after your lenses, put them in and take them out. Always wash and dry your hands thoroughly before putting your lenses in or taking them out.

Never use damaged lenses.

After use and before re-use, the lenses must be taken out, cleaned and decontaminated using the maintenance products your specialist has recommended.

These lenses are designed to be worn by day; put them in the morning and take them out at night.

Always take your lenses out before going to bed.

If you do not wear your lenses for several days, refer to the information about your maintenance solution before re-using them. Always make sure you have your glasses with you (in case you find yourself obliged to take your lenses out).

To avoid any alteration to the material or shape of your lenses, make sure you change them every 18 months. Make sure that you attend the scheduled appointments with your practitioner; these are essential to monitor any changes.

PUTTING IN YOUR LENSES

Handling contact lenses entails a series of simple moves that should be done in respect of basic rules of hygiene.

Before handling, hands must be washed, rinsed and dried on a clean, fluff-free towel.

Put them in and take them out over a clean, flat, clear, well-lit surface.

If you need to handle your lenses over a washbasin, make sure that the plug is in.

To avoid any confusion between the right and left lens, it is best to always proceed in the same order: for example, first put the right lens in and then the left one.

We recommend putting your lenses in after your morning wash and before putting make-up on; remove them after removing make-up and cleansing in the evening.
Putting them in without a suction holder: Stabilise the lens by resting it between your thumb, index finger and middle finger, held tightly together (Figure 1). Completely fill with lens with a solution without preservative.

Caution: never put the scleral lens on dry, without the solution in it.

Angle your head downwards and use your index finger and thumb of the other hand to open your eyelids.

Place the lens on the eye (Figure 2).

Check that there are no air bubbles trapped between the lens and your eye (Figure 3) as they can cause discomfort and poor vision. If so, you will need to remove the lens and then put it in again (using the same procedure).

Repeat for the other eye.

Putting them in with a suction holder:

Stabilise the lens by resting it on a wide-based suction holder (r Place the lens on the eye (Figure 2).

Check that there are no air bubbles trapped between the lens and your eye (Figure 3) as they can cause discomfort and poor vision. If so, you will need to remove the lens and then put it in again (using the same procedure).

TAKING OUT YOUR LENSES

Apply a drop of comfort solution to help remove it and then lean your head over a mirror placed flat on a table.

Removal without a suction holder: With your eyes wide open, place the index finger of one hand on the upper eyelid at the edge of the lashes and the index finger of the other hand on the lower eyelid, at the edge of the lashes. Now bring your two index fingers together, exerting gentle pressure to release the lens from the eye (Figure 4).

Removal with a suction holder:

If using a narrow-based suction holder, dampen it first with the comfort solution.

Pull your lower eyelid down and apply the suction holder to the bottom of the lens.

Caution: never remove the lens with the suction holder at the lens centre.

Use the suction holder to pull the lens from the side of your nose so that an air bubble forms.

Gently remove it by lifting the edge that has now become unstuck.

Now remove the lens from the suction holder, gliding it off gently without grasping its edges.

MAINTENANCE

It is essential that you take care of your lenses. Always follow the instructions given on the maintenance products precisely. Decontaminate your lenses each time you take them out, to eliminate bacteria and be sure that you can continue wearing them comfortably, without risk.

Do not change the maintenance system your specialist has recommended.

Clean your container regularly, following the instructions given on the products used to maintain the lenses.

CONTRAINDICATIONS

Do not take any medications for eyes (eye drops or other ophthalmological products) without consulting your specialist.

Do not clean your lenses or container with tap water.

Do not use your saliva or any other household product to look after your lenses: only use the products recommended by your specialist.

Do not change brand of your maintenance product without consulting your specialist.

Do not allow your lenses to come into contact with lacquer or any other aerosol cosmetic product.

Do not leave your lenses in to sleep (see § precautions for use).

Do not keep your lenses in while having a bath or shower or if you are going to swim in a pool or the sea.

Do not wear your lenses if exposed to dangerous fumes or irritants.

Pregnancy and some medications can cause tearing; talk to your specialist.

Wearing lenses is not recommended when you are suffering from an eye disorder and in very dry atmospheres (aeroplane, hairdressers' hair-drying hood, etc.)

If you suffer from any of the following symptoms: irritated eye, redness, sensation of discomfort, watering eyes, a decline in sharpness of vision or poor sight, etc., take the lens out and clean it. If discomfort persists, contact your specialist.

SIDE EFFECTS

The following possible side effects of lens wearing require you to stop wearing your lenses and consult a specialist:

- Irritated eye, discomfort, watering
- Vascularisation, abrasion, ulceration, abscess
- Photophobia

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