GB Recommendations for use

Guide to the use of your soft lenses

This leaflet contains important information; read it and keep it for future reference. Lenses are medical devices and must always be adjusted by a contact lens specialist. For correct use, it is important that you follow all the instructions given by your specialist and all the recommendations given on the packaging of your lenses and maintenance products.

INSTRUCTIONS:

Soft lenses are corneal lenses for daytime wear, changing them every three months, six months or once a year, to correct short-sightedness, long-sightedness, astigmatism and presbyopia on regular corneas.

RECOMMENDED RENEWAL:

Every three months, every six months or once a year, as applicable.

PACKAGING:

The lenses are packaged in a sterile bottle containing a saline solution. The bottle is fitted with a tamper-proof system until first opening. If packaging is damaged, do not use the lens.

The label provides the following information:

	The lens brand name
	The name of its material
	The name and address of the manufacturer
	
SN	The serial number
	
CE 0459	European conformity with health and safety requirements and certifying organisation number
	Expiry date
STERILE 	Steam sterilisation
	Leaflet consultation
	Do not open if the packaging is damaged

PRECAUTIONS FOR USE

Your specialist will give you precise instructions on how to look after your lenses, put them in and take them out. Always wash and dry your hands thoroughly before putting your lenses in or taking them out.

Never use damaged lenses.

After use and before re-use, the lenses must be taken out, cleaned and decontaminated using the maintenance products your specialist has recommended.

These lenses are designed to be worn by day; put them in the morning and take them out at night.

Always take your lenses out before going to bed.

If you do not wear your lenses for several days, refer to the information about your maintenance solution before re-using them. Always make sure you have your glasses with you (in case you find yourself obliged to take your lenses out).

To avoid any alteration to the material or shape of your lenses, make sure you change them every 3 months, 6 months or year, as indicated on the lens bottle.

Make sure that you attend the scheduled appointments with your practitioner; these are essential to monitor any changes.

PUTTING IN YOUR LENSES

Handling contact lenses entails a series of simple moves that should be done in respect of basic rules of hygiene.

- Before handling, hands must be washed, rinsed and dried on a clean, fluff-free towel.

- Do not keep your nails long.

- Put them in and take them out over a clean, flat, clear, well-lit surface.

- If you need to handle your lenses over a washbasin, make sure that the plug is in.

- We recommend putting your lenses in after your morning wash and before putting make-up on; remove them after removing make-up and cleansing in the evening.

- To avoid any confusion between the right and left lens, it is best to always proceed in the same order: for example, first put the right lens in and then the left one.

- Before you put it in, always check that the lens is not inside out (figures 1 and 2): **place the lens on your index finger to check its shape.**

- Use your middle fingers to open your eyelids

- Angle your face towards the mirror, stare at it and bring the lens close to your eye until it comes in contact with the cornea (figure 3).

- Hold it there for a few seconds and then gently release the eyelids.

- Repeat for your left lens.

- Check your vision after putting them in: both right and left eyes.

TAKING OUT YOUR LENSES

- Look up and lower your lower eyelid with your middle finger.

- Remove the lens from your eye carefully, using the tip of your index finger and your thumb (figure 4).

- Repeat for your left lens.

MAINTENANCE

It is essential that you take care of your lenses. Always follow the instructions given on the maintenance products precisely. Decontaminate your lenses each time you take them out, to eliminate bacteria and be sure that you can continue wearing them comfortably, without risk.

Do not change the maintenance system your specialist has recommended.

Never use maintenance products for hard or oxygen-permeable lenses; this can damage your soft lenses.

Clean your container regularly, following the instructions given on the products used to maintain the lenses.

CONTRAINDICATIONS

Do not take any medications for eyes (eye drops or other ophthalmological products) without consulting your specialist.

Do not clean you lenses or container with tap water.

Do not use your saliva or any other household product to look after your lenses: only use the products recommended by your specialist.

Do not change brand of your maintenance product without consulting your specialist.

Do not allow your lenses to come into contact with lacquer or any other aerosol cosmetic product.

Do not leave your lenses in to sleep (see § Precautions for use).

Do not wear soft lenses when having a bath or shower or swimming in a pool or the sea.

Do not wear your lenses if exposed to dangerous fumes or irritants.

Pregnancy and some medications can cause tearing; talk to your specialist.

Wearing lenses is not recommended when you are suffering from an eye disorder and in very dry atmospheres (aeroplane, hairdressers' hair-drying hood, etc.)

If you suffer from any of the following symptoms: irritated eye, redness, sensation of discomfort, watering eyes, a decline in sharpness of vision or poor sight, etc., take the lens out and clean it. If discomfort persists, contact your specialist.

SIDE EFFECTS

The following possible side effects of lens wearing require you to stop wearing your lenses and consult a specialist:

- Irritated eye, discomfort, watering
- Vascularisation, abrasion, ulceration, abscess
- Photophobia

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