# AQUADROP+

**F**

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**GB**

**US**

## Confort ocular drops based upon Sodium Hyaluronate

**AQUADROP+** is a sterile solution for ophthalmic use with a physiological pH, containing sodium hyaluronate (0,4%) and distilled waters of ginkgo biloba, bilberry, fennel and centella, boric acid, sodium tetraborate and sodium chloride, dispensed in practical single-dose containers of 0,5 ml, preservative free.

**AQUADROP+** performs a moisturizing and lubricating action on the ocular surface and give a temporary relief of burning, irritation and discomfort due to dryness of the eye.

It can be used by contact lens wearers without the need of removing the lenses. The device is also suitable for children, after doctor consultation.

### INTENDED USE

- Insertion and removal of orthokeratology contact lenses.

**AQUADROP+** can be used as a protectant against further irritation of the eye due to:

- Environmental factors (air conditioned, excessive exposure to solar radiation, wind, smoke, pollution).
- Dryness of the eyes due to excessive fatigue caused by working on the computer for lengthy periods, study or dry eye syndrome.
- Ophthalmic stress following surface eye surgery.

- Wearing of contact lenses many hours a day.

### INSTRUCTIONS FOR USE

Detach the single-dose container from the strip and open the single-dose container by turning and then lifting off the closing cap.

Instill 1 or 2 drops in each eye by gently squeezing the single dose.

After use immediately close the single-dose container: the content must be used within 12 hours after the first opening. Storage before 30°C. On each single dose are reported the variable data: batch number and expiry date.

### PRECAUTIONS

The product is for ophthalmic use only - Do not use the product if the container is damaged or not perfectly closed when you open it for the first time - During application, do not touch the eye or any other surface with the container's dispensing edge - Do not use in case of known intolerance toward any of the components - In case of side effects during use, interrupt the treatment and consult a doctor.

### STORAGE CONDITIONS

✂ Storage below 30°C - Single doses must be used within 12 hours after the first opening.

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